OTOSAN[®]

Spray[®]

con estratti Bio

OTOSAN[®]

Spray[®]

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OTOSAN[®]

Spray[®]

con extractos Biológicos

OTOSAN[®]

Spray[®]

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OTOSAN[®]

Spray[®]

with Organic extracts

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Natural, non-invasive cleansing

Otosan Spray[®] with organic plant extracts delivers delicate but effective action for cleaner, healthier ears because it eliminates excess wax from the ear canal and, if used regularly, it prevents wax formation.

Highly effective natural ingredients

Otosan Spray[®] with organic plant extracts is the result of cutting-edge medical science research. Free from colourings and preservatives, it guarantees maximum levels of tolerance. Combines the effective cleansing action of saline solution with the antiseptic properties of tea tree oil. The moisturising action of aloe combines with the other properties of the 100% organically grown plant extracts: lemon with outstanding purifying properties, soothing camomile, the effective anti-inflammatory action of marigold and the emollient power of mallow.

How to use

With your head straight, bring the **Otosan Spray[®]** up to the ear so that the spray nozzle is in contact with the ear canal. Press the spray button a couple of times keeping the spray container upright.

Wait a few minutes to allow **Otosan Spray[®]** to do its work. Lean your head over to the side of the treated ear to let the liquid and impurities flow out. Clean and dry the ear gently with a tissue.

Repeat the treatment on the other ear. Rinse the spray nozzle with warm water after each use.

When to use

Otosan Spray[®] is not the only wax-removing system, however, its preventative action helps keep the ears clean and healthy. We recommend using it once a day.

Warnings

Use at room temperature (20°C-25°C). Do not expose the bottle to sources of heat. Do not use if there is damage or injury to the ear drum or canal, any ear related medical conditions, after recent surgery or in the case of allergies to any of the ingredients. Keep out of the reach of children. Do not use on children under 30 months of age. For external use only. Avoid contact with the eyes. Not to be used after the expiry date.

OTOSAN SPRAY, OTOSAN CONES AND OTOSAN DROPS
a complete range of natural products
for cleaner, healthier ears